

Fascin

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

DESCRIPTION : Human fascin is a highly conserved actin-bundling protein. Fascin, encoded by the human homolog for sn (hsn) gene, has been localized to microspikes and stress fibers of cultured cells where it is thought to be involved in the formation of microfilament bundles. It is expressed predominantly in dendritic cells. Lymphoid cells, myeloid cells and plasma cells are negative. However, Reed Sternberg cells in Hodgkin's lymphoma are positive for fascin staining. Epstein-Barr virus may induce exp. of fascin in B cells.

CATALOG NO : PL1112	PL1112-R7	7 ML RTU 70 TEST
	PL1112-R1	1 ML RTU 10 TEST
STAINING PATTERN : Cytoplasmic	PL1112-1	1 ML 1/500 5000 TEST
	PL1112-0,1	0,1 ML 1/500 500 TEST

POSITIVE CONTROL : Hodgkin's lymphoma or spleen

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Mouse

CLONE : FCN01

ANTIBODY CONCENTRATION : Not known.

SPECIES REACTIVITY : Human and Rat. Others-not known

EPITOPE : Not determined

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 30 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).