

Cytomegalovirus (CMV)

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Rabbit Monoclonal.

DESCRIPTION :

Cytomegalovirus (CMV) is a virus of the Herpes-virus group; in humans it is commonly known as HCMV or Human Herpesvirus 5 (HHV-5). CMV belongs to the Betaherpesvirinae subfamily of Herpesviridae, which also includes Roseolovirus.

PL717-0,1 0,1 ML 1/100 100 TEST POSITIVE CONTROL : CMV Infected Tissue VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.) HOST : Mouse CLONE : 8B1.2 ANTIBODY CONCENTRATION : Not known SPECIES REACTIVITY : Human EPITOPE : Not known MICROBIOLOGICAL STATE : This product is not sterile. PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).	CATALOG NO : PL717 STAINING PATTERN : Nuclear	PL717-R7 PL717-R1 PL717-1	7 ML RTU 70 TEST 1 ML RTU 10 TEST 1 ML 1/100 1000 TEST
stabilizing protein and 0.015mol/L sodium azide.) HOST: Mouse CLONE: 8B1.2 ANTIBODY CONCENTRATION: Not known SPECIES REACTIVITY: Human EPITOPE: Not known MICROBIOLOGICAL STATE: This product is not sterile. PRETREATMENT: Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME: 60 minutes at Room Temperature STAINING TIPS: If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY: This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.	POSITIVE CONTROL : CMV Infected Tissue	PL717-0,1	0,1 ML 1/100 100 TEST
 CLONE: 8B1.2 ANTIBODY CONCENTRATION: Not known SPECIES REACTIVITY: Human EPITOPE: Not known MICROBIOLOGICAL STATE: This product is not sterile. PRETREATMENT: Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME: 60 minutes at Room Temperature STAINING TIPS: If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY: This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	stabilizing protein and 0.015mol/L sodium azide.)		
 ANTIBODY CONCENTRATION : Not known SPECIES REACTIVITY : Human EPITOPE : Not known MICROBIOLOGICAL STATE : This product is not sterile. PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	HOST : Mouse		
 SPECIES REACTIVITY : Human EPITOPE : Not known MICROBIOLOGICAL STATE : This product is not sterile. PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	CLONE : 8B1.2		
 EPITOPE : Not known MICROBIOLOGICAL STATE : This product is not sterile. PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	ANTIBODY CONCENTRATION : Not known		
 MICROBIOLOGICAL STATE: This product is not sterile. PRETREATMENT: Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME: 60 minutes at Room Temperature STAINING TIPS: If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY: This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	SPECIES REACTIVITY : Human		
 PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	EPITOPE : Not known		
 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min. PRIMARY ANTIBODY INCUBATION TIME: 60 minutes at Room Temperature STAINING TIPS: If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY: This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 	MICROBIOLOGICAL STATE : This product is not sterile.		
 STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user. 			
too strong, check pretreatment, use higher dilution or shorter time. STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.	PRIMARY ANTIBODY INCUBATION TIME: 60 minutes at Room Temperature		
at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.			
TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).	at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent		

