

Toluidine Blue

INTENDED USE : For In Vitro Diagnostic Use

Catalog No	Volume
PL-050-TOL	50 ml
PL-100-TOL	100 ml
PL-500-TOL	500 ml
PL-1000-TOL	1000 ml

DESCRIPTION

Staining of acid substances, mast cell granules, acid substance of cartilage and some epithelial mucins.

STORAGE & SHELF LIFE

Store at room temperature. Stable for minimum 3 years.

MICROBIOLOGICAL STATE

Product(s) not sterile.

MATERIALS REQUIRED BUT NOT PROVIDED

N/A

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE

- 1) Sections to water
- 2) Toluidine Blue O, 1 - 2 minutes
- 3) Wash in distilled water
- 4) Cover directly with coverslide

RESULTS

Metachromatic substances : Different blue tonalities

Troubleshooting

Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).