

Catenin, beta

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

DESCRIPTION : The catenins (alpha, Beta and gamma) are ubiquitously expressed, cytoplasmic proteins associated with E-cadherin at cellular junctions. Beta-catenin also binds to N-cadherin and co-immunoprecipitates with APC. Cadherin/catenin complexes are linked to the cytoskeleton via a direct association between alpha-actinin and alpha-catenin. Increases tyrosine phosphorylation can disrupt catenin-cadherin complexes, influencing cellular adhesion.

CATALOG NO : PL9035	PL9035-R7	7 ML RTU 70 TEST
	PL9035-R1	1 ML RTU 10 TEST
STAINING PATTERN : Cytoplasmic and cell membrane	PL9035-1	1 ML 1/100 1000 TEST
	PL9035-0,1	0,1 ML 1/100 100 TEST

POSITIVE CONTROL : MCF-7 cells. Breast CA

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Rabbit

CLONE : Polyclonal

ANTIBODY CONCENTRATION : 200ug / ml

SPECIES REACTIVITY : Human and Mouse. Others not tested

EPITOPE : C-terminal

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 30 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).