

## Crystal Violet % 0,5 Aqueous

**INTENDED USE** : For In Vitro Diagnostic Use

<u>Catalog No</u>	<u>Volume</u>
PL-050-VIO	50 ml
PL-100-VIO	100 ml
PL-500-VIO	500 ml
PL-1000-VIO	1000 ml

### **DESCRIPTION**

Staining of acid substances, mast cell granules, acid substance of cartilage and some epithelial mucins.

### **STORAGE & SHELF LIFE**

Store at room temperature. Stable for minimum 3 years.

### **MICROBIOLOGICAL STATE**

Product(s) not sterile.

### **MATERIALS REQUIRED BUT NOT PROVIDED**

N/A

### **SPECIMEN & REAGENT PREPARATION**

Refer to Procedure.

### **PROCEDURE**

- 1) Sections to water
- 2) Crystal violet, 5 minutes
- 3) Acetic acid 0.5%, 15 seconds
- 4) Wash in tap water
- 5) Mount in aqueous mount

### **RESULTS**

Metachromatic substances : Different blue tonalities

### **Troubleshooting**

Please contact Patolab Technical Support by e-mail ([patolab@patolab.com.tr](mailto:patolab@patolab.com.tr)).