

## Neutral Red % 0,5 Aqueous

**INTENDED USE** : For In Vitro Diagnostic Use

<b><u>Catalog No</u></b>	<b><u>Volume</u></b>
PL-500-RED	500 ml
PL-1000-RED	1000 ml

### **DESCRIPTION**

Product for the preparation of cyto-histological samples for optical microscopy. Red nuclear stain. Excellent for methods such as Alcian Blue and Perls.

### **STORAGE & SHELF LIFE**

Store at room temperature. Stable for minimum 3 years.

### **MICROBIOLOGICAL STATE**

Product(s) not sterile.

### **MATERIALS REQUIRED BUT NOT PROVIDED**

N/A

### **SPECIMEN & REAGENT PREPARATION**

Refer to Procedure.

### **PROCEDURE :**

- 1) Sections to water
- 2) Neutral red 5 minutes
- 3) Wash in distilled water
- 4) Dehydrate
- 5) Clearing agent and mount

### **RESULTS**

Nuclei : Red

### **Troubleshooting**

Please contact Patolab Technical Support by e-mail ([patolab@patolab.com.tr](mailto:patolab@patolab.com.tr)).