

Chromogranin A

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. It is Rabbit Monoclonal.

DESCRIPTION : Chromogranin A (a protein of 439 amino acid which is encoded on chromosome 14) is present in neuroendocrine cells throughout the body, including the neuroendocrine cells of the large and small intestine, adrenal medulla and pancreatic islets. It is an excellent marker for carcinoid tumors, pheochromocytomas, paragangliomas, and other neuroendocrine tumors. Coexpression of chromogranin A and neuron specific enolase (NSE) is common in neuroendocrine neoplasms.

CATALOG NO :	PL9112	PL9112-R7	7 ML RTU 70 TEST
		PL9112-R1	1 ML RTU 10 TEST
STAINING PATTERN :	Cytoplasmic	PL9112-1	1 ML 1/200 2000 TEST
		PL9112-0,1	0,1 ML 1/200 200 TEST
POSITIVE CONTROL :	Pancreas.		

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Rabbit

CLONE : SP12

ANTIBODY CONCENTRATION : Not known

SPECIES REACTIVITY : Human. Others not tested.

EPITOPE : Not determined

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 30 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).