

Alcian Blue 8GX

INTENDED USE : For In Vitro Diagnostic Use

Catalog No	Volume
PL-050-ALB	50 ml
PL-500-ALB	500 ml
PL-1000-ALB	1000 ml

DESCRIPTION

Product for the preparation of cyto-histological samples for optical microscopy.
Reagent for evidencing acid mucopolysaccharides.

STORAGE & SHELF LIFE

Store at room temperature. Stable for minimum 3 years.

MICROBIOLOGICAL STATE

Product(s) not sterile.

MATERIALS REQUIRED BUT NOT PROVIDED

N/A

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE for PAS Staining :

- 1) Sections to water
- 2) **Alcian blue pH 2.5** 30 minutes
- 3) Sodium tetraborate solution 10 minutes
- 4) Wash in distilled water
- 5) Dehydrate
- 6) Clearing agent and mount

RESULTS

Acid mucopolysaccharides : Turquoise blue

Troubleshooting

Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).