

Galectin-3

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

DESCRIPTION : Galectin-3 is a member of the beta-galactosidase-binding lectin family. It is associated with cell growth, adhesion, inflammation, mRNA processing, and apoptosis. Aberrant expression of Galectin-3 is related to malignant transformation and metastasis in carcinomas of the breast, colon and thyroid. Galectin-3 reactivity can be seen in the nucleus of neutrophils, vascular endothelium, carcinomas of the colon, breast, and thyroid.

CATALOG NO :	PL1756	PL1756-R7	7 ML RTU 70 TEST
		PL1756-R1	1 ML RTU 10 TEST
STAINING PATTERN :	Cytoplasmic	PL1756-1	1 ML 1/100 1000 TEST
		PL1756-0,1	0,1 ML 1/100 100 TEST

POSITIVE CONTROL : Papillary or follicular thyroid carcinoma

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Mouse

CLONE : 9C4

ANTIBODY CONCENTRATION : Not known

SPECIES REACTIVITY : Human. Others not tested.

EPITOPE : Not determined

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 30 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).