

## CD2

## IN VITRO DIAGNOSTIC DATASHEET

**INTENDED USE :** IN VITRO DIAGNOSTIC USE

This Rabbit Monoclonal product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

**DESCRIPTION :**

CD2 is mainly present on the surface of mature T cells and NK cell membranes, while B cells are usually not expressed. It can be used to label the diagnosis and study of normal T cells and their associated tumors, such as pre-t cell lymphoma, marginal T cell lymphoma, and anaplastic cell lymphoma. CD2 antibody is helpful for the identification of precursor and mature T-cell lymphomas. Aberrant loss of CD2 in T-cell lymphomas may help to distinguish them from reactive T-cell proliferation.

<b>CATALOG NO :</b>	PL1002	PL1002-R7	7 ML RTU 70 TEST
		PL1002-R1	1 ML RTU 10 TEST
<b>STAINING PATTERN :</b>	Cell Surface	PL1002-1	1 ML 1/100 1000 TEST
		PL1002-0,1	0,1 ML 1/100 100 TEST
<b>POSITIVE CONTROL :</b>	Tonsil		

**VOLUME :** 7 ml Ready to Use ( 7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide. )

**HOST :** Rabbit

**CLONE :** PL-5

**ANTIBODY CONCENTRATION :** Not known

**SPECIES REACTIVITY :** Human. Others not tested.

**EPITOPE :** Not known

**MICROBIOLOGICAL STATE :** This product is not sterile.

**PRETREATMENT :** Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

**PRIMARY ANTIBODY INCUBATION TIME :** 60 minutes at Room Temperature

**STAINING TIPS :** If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

**STORAGE AND STABILITY :** This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

**TROUBLESHOOTING :** Please contact Patolab Technical Support by e-mail ( patolab@patolab.com.tr ).