

PEPSIN ENZYME

INTENDED USE:

For In Vitro Diagnostic Use

Catalog No	Volume
PL-015-PE	15 ml (150 Slides)
PL-125-PE	125 ml (1250 Slides)

DESCRIPTION

Pepsin is a commonly used digestive enzyme for immunohistochemical procedures. This product is provided as a single component, ready-to-use solution. The pH of the pepsin solution is 2.0.

WARNINGS & PRECAUTIONS

Refer to MSDS.

STORAGE & SHELF LIFE

Store at 2-8°C. Each component is stable for 24 months.

MICROBIOLOGICAL STATE

Product(s) not sterile.

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE

- 1. Incubate tissue section for 10 minutes at 37°C.
- 2. Rinse and proceed with immunostaining.

Troubleshooting

Please contact PatoLab Technical Support by e-mail (patolab@patolab.com.tr).

PatoLab Biyomedikal Cihazlar ve Sarf Malzemeler San. ve Tic. Ltd. Şti.patolab@patolab.com.trwww.patolab.com.tr



IVD