

p16INK4a

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This Mouse Monoclonal product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

DESCRIPTION : p16INK4a is a tumor suppressor protein. It is a specific inhibitor of cdk4/cdk6, and a tumor suppressor involved in the pathogenesis of a variety of malignancies. Recent analyses of the p16INK4a gene revealed homozygous deletions, nonsense, missense, or frameshift mutations in several human cancers. Although the frequency of p16INK4a abnormalities is higher in tumor-derived cell lines than in unselected primary tumors, significant subsets of clinical cases with aberrant p16INK4a gene have been reported among melanomas, gliomas, esophageal, pancreatic, lung, and urinary bladder carcinomas, and some types of leukemia.

CATALOG NO : PL2016	PL2016-R7	7 ML RTU 70 TEST
	PL2016-R1	1 ML RTU 10 TEST
	PL2016-1	1 ML 1/100 1000 TEST
STAINING PATTERN : Cytoplasmic and nuclear	PL2016-0,1	0,1 ML 1/100 100 TEST

POSITIVE CONTROL : Cervical squamous cell carcinoma

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Mouse

CLONE : G175-405

ANTIBODY CONCENTRATION : Not known

SPECIES REACTIVITY : Human. Others not tested.

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 60 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact PatoLab Technical Support by e-mail (patolab@patolab.com.tr).