

## Galectin-3

**INTENDED USE :** IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

**DESCRIPTION :** Galectin-3 is a member of the beta-galactosidase-binding lectin family. It is associated with cell growth, adhesion, inflammation, mRNA processing, and apoptosis. Aberrant expression of Galectin-3 is related to malignant transformation and metastasis in carcinomas of the breast, colon and thyroid. Galectin-3 reactivity can be seen in the nucleus of neutrophils, vascular endothelium, carcinomas of the colon, breast, and thyroid.

<b>CATALOG NO :</b>	PL1756	PL1756-R7	7 ML RTU 70 TEST
		PL1756-R1	1 ML RTU 10 TEST
<b>STAINING PATTERN :</b>	Cytoplasmic	PL1756-1	1 ML 1/100 1000 TEST
		PL1756-0,1	0,1 ML 1/100 100 TEST

**POSITIVE CONTROL :** Papillary or follicular thyroid carcinoma

**VOLUME :** 7 ml Ready to Use ( 7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide. )

**HOST :** Mouse

**CLONE :** 9C4

**ANTIBODY CONCENTRATION :** Not known

**SPECIES REACTIVITY :** Human. Others-not known.

**EPITOPE :** Not determined

**MICROBIOLOGICAL STATE :** This product is not sterile.

**PRETREATMENT :** Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

**PRIMARY ANTIBODY INCUBATION TIME :** 30 minutes at Room Temperature

**STAINING TIPS :** If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

**STORAGE AND STABILITY :** This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

**TROUBLESHOOTING :** Please contact Patolab Technical Support by e-mail ( patolab@patolab.com.tr ).