

## Myogenin

## IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE: IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

**DESCRIPTION:** Myogenin is a member of a family of myogenic regulatory genes, which includes MyoD, myf5 and MRF4. These genes encode a set of transcription factors, which are essential for muscle development. Transfection of myogenin into multipotential mesodermal cells have been shown to convert the mesodermal cells to myoblasts.

**CATALOG NO:** PL1113 PL1113-R7 7 ML RTU 70 TEST PL1113-R1 1 ML RTU 10 TEST

**STAINING PATTERN**: Nuclear PL1113-1 1 ML 1/400 4000 TEST PL1113-0,1 0,1 ML 1/400 400 TEST

POSITIVE CONTROL: Rhabdomyosarcoma

VOLUME: 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing

stabilizing protein and 0.015mol/L sodium azide.)

**HOST:** Mouse

CLONE: F5D

**ANTIBODY CONCENTRATION:** Not known

**SPECIES REACTIVITY:** Human

EPITOPE: Not known

**MICROBIOLOGICAL STATE:** This product is not sterile.

**PRETREATMENT**: Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM Citrate, pH 6.0, for 10-20 min followed by cooling at RT for 20 min.

PRIMARY ANTIBODY INCUBATION TIME: 30 minutes at Room Temperature

STAINING TIPS: If the staining is too light, use lower dilution or longer time. If the staining is

too strong, check pretreatment, use higher dilution or shorter time.

**STORAGE AND STABILITY:** This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent

is not stored as recommended, performance must be validated by the user.

**TROUBLESHOOTING:** Please contact Patolab Technical Support by e-mail ( patolab@patolab.com.tr ).



