

PatoMount Mounting Medium

INTENDED USE : For In Vitro Diagnostic Use

<u>Catalog No</u>	<u>Volume</u>
PL-050-MOU	50 ml
PL-125-MOU	125 ml
PL-500-MOU	500 ml
PL-1000-MOU	1000 ml

DESCRIPTION

PatoMount Mounting Medium is used to stick the slide and coverslip.

STORAGE & SHELF LIFE

Store at room temperature. Stable for minimum 3 years.

MICROBIOLOGICAL STATE

Product(s) not sterile.

MATERIALS REQUIRED BUT NOT PROVIDED

N/A

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE

- 1) Place 1 drop of mounting medium to the slide.
- 2) Place coverslip on the Slide.
- 3) Be sure not to have bubbles.

Troubleshooting

Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).