

Growth hormone

IN VITRO DIAGNOSTIC DATASHEET

INTENDED USE : IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

DESCRIPTION : Growth hormone (also known as GH) is a 191 amino acid, single-chain polypeptide hormone which is synthesized, stored and secreted by the somatotroph cells within the lateral wings of the anterior pituitary gland, which stimulates growth and cell reproduction in humans and other animals. GH antibody is a useful marker in classification of pituitary tumors and the study of pituitary disease (acromegaly). It reacts with GH-producing cells.

CATALOG NO :	PL1328	PL1328-R7	7 ML RTU 70 TEST
		PL1328-R1	1 ML RTU 10 TEST
STAINING PATTERN :	Cytoplasmic	PL1328-1	1 ML 1/2000 20000 TEST
		PL1328-0,1	0,1 ML 1/2000 2000 TEST
POSITIVE CONTROL :	Normal Pituitary		

VOLUME : 7 ml Ready to Use (7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide.)

HOST : Mouse

CLONE : EP267

ANTIBODY CONCENTRATION : Not known

SPECIES REACTIVITY : Human. Others-not known.

EPITOPE : Not determined.

MICROBIOLOGICAL STATE : This product is not sterile.

PRETREATMENT : Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0 for 10-20 minutes followed by cooling at room temperature for 20 min.

PRIMARY ANTIBODY INCUBATION TIME : 30 minutes at Room Temperature

STAINING TIPS : If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

STORAGE AND STABILITY : This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

TROUBLESHOOTING : Please contact Patolab Technical Support by e-mail (patolab@patolab.com.tr).